SPECTRA Disposable Radio Frequency (RF) Needle	
English	Instruction for Use
Indications for Use: The SPECTRA Disposable Radio Frequency (RF) Needles are indicated for use in radio frequency heat lesion procedures for relief of pain.	
Instructions for Use:	
1	Remove the needle guard.
2	Read and follow all instructions for use when placing a Spectra Disposable Radio Frequency (RF) Needle for the subsequent medical procedure.
3	The physician will place the cannula under fluoroscopic (C-Arm) guidance. Once the cannula has been positioned the stylet may be removed.
4	The thermocouple (TC) or thermistor (TM) electrode can be inserted into the needle.
5	Follow the Manufacturer's instructions for use for the TC electrode and RF generator.
6	When the procedure is complete, remove the TC or TM and dispose of per Manufacturer's instructions.
7	Then carefully remove the Needle and place it in a Sharps Container.

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0173-001-04_SMD-IFU-0009 Rev 005

SPECTRA	
	Warnings:
<ul style="list-style-type: none"> This product is for single use, DO NOT REUSE - DO NOT RESTERILIZE. Reuse and/or repackaging may create a risk of patient or user infection, compromise the structure integrity and/or essential material and design characteristics of the device, which may lead to device failure, and/or injury, illness or death of the patient. Do not use if package is opened or damaged. Do not alter this device or Straighten Bend Needle – Discard if Bent. DO NOT AUTOCLAVE. This device may cause interference in proximity of a magnetic resonance imaging machine. Handle and discard in accordance with accepted medical practice and applicable regulations. After use this product is a potential biohazard. Re-sheathing of needles is hazardous. 	
Cautions:	<ul style="list-style-type: none"> Carefully read and follow all relevant hospital/medical institutional guidelines when placing a Radio Frequency (RF) Needle for the subsequent medical procedure. Care and adherence to universal precautions must be taken to avoid accidental needle stick injuries. Only qualified healthcare practitioners with proper training should insert, manipulate and remove these devices. Follow standard infection control precautions as specified by the CDC (USA) or local equivalent. Federal (U.S.A.) Law restricts this device to sale by or on the order of a physician.
NOTE: Refer to the IFU from the manufacturer of the thermocouple (TC) or thermistor (TM) and the RF Generator with which the Spectra Disposable Radio-Frequency Needle is being used for additional WARNINGS and CAUTIONS.	

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0173-001-04_SMD-IFU-0009 Rev 005

SPECTRA Disposable Radio Frequency (RF) Needle	
English	ADVERSE EVENTS
1	Potential complications from procedure with which this Device is used include, but are not limited to: Mild Discomfort; Temporary Pain; Paresthesia; Local Pain; Numbness of Back; Dyspareunia; Soreness; Increased intensity of low back pain; Intracranial Hypotension; Bradycardia; Transient Thermal sciatic neuropaxia; Inflammation; Disc prolapse.
2	Potential device problems related to the RF Needle: Break / Fracture / Crack / de fective; Material Separation / peeled / delaminated / Erosion / Fragmentation / discolored / corroded / protrusion / extrusion; Device misassembled during shipping / Detachment of device component; Contamination problem; Leak / Splash.

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